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D - WP

Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

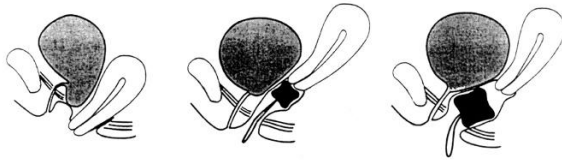
Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Cube Pessary

Application: Different degrees of vagina and uterine prolapse can be treated with the cube pessary. The vacuum effect achieves a good adhesion so that this pessary can even be used in cases when the pelvic floor is not strong enough to support the common ring pessaries. Cube pessaries are also suitable for relaxation of scars, for extending vaginal obstructions and to treat problems with miction or cohabitation. In addition, the pessaries are used before prolapse operations in order to improve the tissue circulation (application with estriol cream). The flexibility of the material facilitates self treatment.

Size: The length of the edge of the cube determines its size ranging from minimum 25mm edge length (size 0) up to maximum 45mm edge length (size 5). The right choice of the cube by the consulting physician is a prerequisite for good adhesion. The cube should be large enough to remain in its position, on the other hand, a cube which is too big, may press on the bladder or intestine and is more difficult to be removed. Treatment should be started with a cube which just adheres also while coughing and pressing. A correction may be necessary after a few days or weeks. In cases with prolapse rather small models, in cases with a constricted vagina larger models should be chosen. In cases with isolated prolapse of the uterus a small pessary will be placed in the upper third of the vagina (Figure centre), in cases with an additional prolapse of the bladder and intestine a larger pessary will be placed rather in the middle third of the vagina (Figure right).

Use: In general, cube pessaries are changed by the patient herself, after having been instructed by a physician or a nurse. **The cube should be removed daily in the evening and inserted again in the morning.** Thus the vaginal tissue may recover. In patients who are planned for a vaginal operation it may make sense to leave the pessary to improve the estrogen effect. In cases with a large prolapse, one can use a smaller pessary over night. Irrespective of the age of the patient, it is of advantage to spread an estriol cream on 2 or 3 edges of the cube before insertion. This will aid in the insertion, but also improve the tissue circulation and recovery of a vaginal flora. Since one needs a relatively large amount of cream (large surface) to reach a gliding effect, we recommend the use of a low dispensed cream with 0.1 mg estriol/1 g cream, in rare cases with 0.5 mg estriol/1 g cream.

For insertion of the cube the woman may put one foot on a stool or the edge of a bed; if this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying and thereby to push the cube as far as possible up the vagina.

For removing the cube the woman has to pull the cube by its thread to varying directions, to resolve the vacuum effect until a resistance is felt by the pelvic floor muscles. With the forefinger and middle finger the patient may loosen the edges of the cube to completely remove the vacuum effect and pull the pessary further down.

Side effects/Complications: If the thread should exceptionally tear, the pessary has to be removed by the physician. In rare case with a large cystocele the cube pessary may manifest a stress incontinence, which would then signify the same effect by pelvic floor suspension surgery. In this case, the pessary should be turned with the edge into the region between bladder and urethra, or another model (urethra or urethra bowl pessary) should be preferred. Contraindications for estriol creams should be considered (e.g. pregnancy, breastfeeding, estrogen-dependent tumors). If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. Before each insertion the surface of the pessary must be inspected. In case of clefts the pessary must be exchanged. Cube pessaries with buttons must be inserted with the button pointing towards the uterus and the thread towards the vagina. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar).

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