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Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Hodge Pessary

Application: The Hodge pessary was introduced as a stiff ring bended in two dimensions, whereby the „elevation of the uterus“ was the primary aim, an indication which has lost its significance. Our Hodge pessary consists of silicone with a half-stiff aluminum filling thus being comfortable and allowing for various shaping and adapting to individual circumstances. **The Hodge pessary is meanwhile only used in cases with incontinence, if ring pessaries are not tolerated because of anatomical aberrations mostly caused by earlier operations.**

Sizes: The Hodge pessary is characterized by the ring shaped diameter, sizes ranging from 55 to 95mm diameter are offered. The final shape may be about 10mm longer than the diameter.

Use: After spreading of the labia, the Hodge pessary is inserted in an oblique diameter with the wide part first. The cervix should lie in front of the end of the pessary, the concave bend should be directed ventrally. The rear part of the Hodge pessary should surround the cervix. For the isolated treatment of urine incontinence the pessary is turned by 180 degrees in its longitudinal axis, so that the upwards bended end will support the urethra. During the first gynecological examination the physician can bend the end of the pessary, especially the part which lies under the urethra, and thus adapt it to the individual situation for elevation of the urethra. The patients should be trained to change the pessary themselves. It is recommended that the patient removes the pessary in the evening and inserts it in the morning. A coating with estriol cream (short adhesion to the receptors) facilitates the insertion and improves the local blood circulation and epithelium and tissue formation. Since relatively small amounts of cream are needed for achieving the gliding effect, we recommend to use creams with a higher concentration of around 1 mg estriol / 1 g cream.

For the change of the pessary the woman may put one foot on a stool or the edge of a bed, in a position comparable to the insertion of a tampon; if this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying. The patient should be trained how to insert the pessary and how to remove it (surrounding of the ring with her forefinger, edging and loosening of the pessary). The fixation of a pulling thread may be additionally helpful.

Side Effects/ Complications: The daily change avoids complications. Stress incontinence usually does not need any therapy during the night. If any incontinence problems remain, despite having changed the sizes and the forms and despite careful training, the diagnosis and choice of the pessary have to be reconsidered. If necessary one has - at least temporarily - to change to a cube pessary and local estriol cream in order to loosen the scars. Contraindications for estriol creams should be considered (e.g. pregnancy, lactation period, estrogen-dependent tumors).

If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyde-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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Dr. Arabin GmbH & Co KG im FEZE